


Scaler Tip Installation + Maintenance

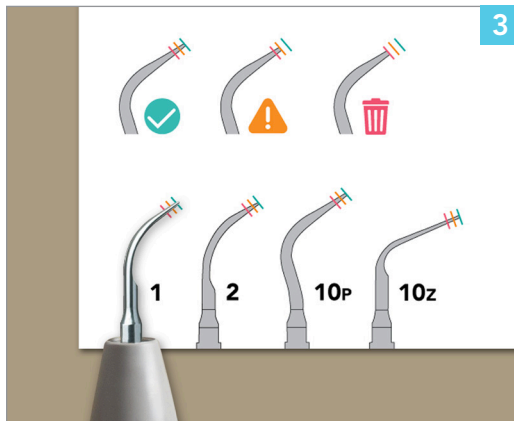


1. Make sure Midmark 1000 power switch is OFF.
2. Screw scaler tip on and tighten moderately with tool provided (**image 1**).
3. Plug handpiece into hose connection, place in designated handpiece holder.
4. Turn Midmark 1000 ON.
5. Set scaler intensity knob to lowest position in the color range that corresponds to the tip color (**image 2**).
6. Verify that the air control pin/knob for the scaler is open (counterclockwise).
7. Remove scaler from holder, depress foot pedal and adjust the water control knob until a steady FINE mist is produced.
8. Scaler is ready to operate.

NOTE: Use the provided Tip Card Wear Guide (**image 3**) to periodically check the wear of the tip. Replace the scaler with a new tip before it reaches the RED line.

Replacement copies of the Tip Card Wear Guide (016-1589-00) are available for purchase.

Scaler Tip Guide

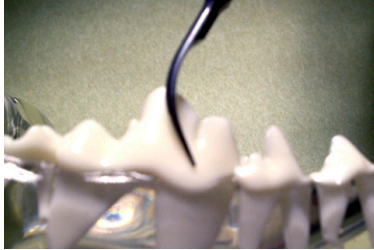


No. 1 Tip: "Universal"

Recommended for treating simple cases and gross **supra-gingival** scaling. Tangential orientation with respect to the tooth surface. To-and-fro sweeping motion to "push" and "pull" calculus without damaging the enamel.



No. 2 Tip: "Voluminous calculus"

Recommended for removing voluminous supra-gingival deposits. **Apply flat end to surface of teeth.**

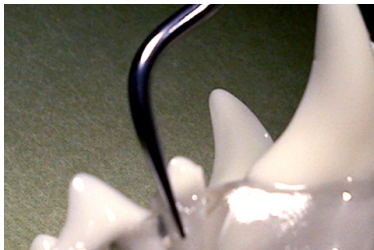

No. 10P Tip: "Shallow pockets"

Fine tip designed for scaling **shallow pockets (<2-3mm)**. It provides greater irrigation than the other tips.