

Aesculap® Clippers Isis & Vega

The reliable partner for your vet clinic!


Aesculap® Isis and Vega

Features


Compact



Quiet

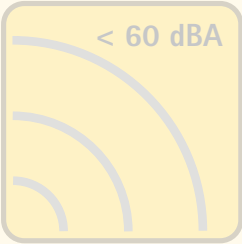

Low maintenance

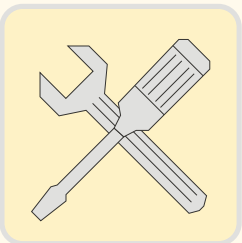

Made in Germany



B. Braun Vet Care


Aesculap®

Clippers Isis and Vega


GT420 Isis
GT420G Isis UK Version

Power unit	DC motor approx. 5700 strokes/min
Charging time	approx. 60 min.
Running time	up to 40 min.
Cutting system	special Aesculap blades
Cutting length	approx. 1.5 mm
Cutting width	24 mm
Weight	approx. 120 g
Charging indicator	in charging station
Set consists of	1 shearing machine, 1x snap-on comb, 1 brush, 1 oil bottle, 1 charging station with tool, 1 user manual

Noise level < 60 dBA

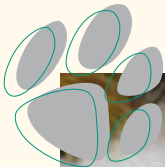

for incisions and biopsies

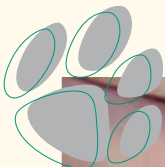

in hard-to-reach places



...for professional wound preparation!

GT410 Vega
GT410G Vega UK Version

Ultrasound preparation



Equine venipuncture



Power unit	DC motor approx. 5700 strokes/min
Charging time	approx. 60 min. quick charge
Running time	up to 100 min.
Cutting system	special Aesculap blades
Cutting length	3 lengths of adjustment: 0.5; 1.5 and 2.5 mm
Cutting width	40 mm
Weight	<250 g
Charging indicator	on the clipper
Set consists of	1 shearing machine, 5 snap-on combs, 1 brush, 1 oil bottle, 1 charging station, 1 user manual
Noise level	< 60 dBA

Cleaning and care



Removing the Vega blade set

Hold the blade set sideways and push it off the front of the trimmer. When replacing the blade set, insert the two catches into the housing and click the blade set into place.



Cleaning the blade set

Use a brush to carefully clean the blade set and remove remnants of hair.



Oiling the blade set

The blades need lubrication after each cleaning. Apply a drop of oil (GT604) to the middle and both ends.



Removing the Isis blade set

Remove the screw using the hex wrench from the bottom of the charger. Take off the shaving head. Perform cleaning and lubrication as described above. Click blade set into place and tighten screw.

Note:
Blood, pus and wound secretion can adhere to the blade. Please clean and oil the blade after each use.

B. Braun Vet Care GmbH | Am Aesculap-Platz | 78532 Tuttlingen | Germany
Fax +49 (0) 74 61 715 12 | info@bbraun-vetcare.de | www.bbraun-vetcare.de

Aesculap – a B. Braun company

The main product trademark 'Aesculap' is a registered trademark of Aesculap AG.

Subject to technical changes. All rights reserved. This brochure may only be used for the exclusive purpose of obtaining information about our products. Reproduction in any form partial or otherwise is not permitted.

V08402

1113/0.5/2